





# Sani-Cloth<sup>®</sup> AF is a two in one solution ideal for cleaning and disinfecting.

- Ready to use: pre-dosed, alcohol free, low linting disposable cleaning and disinfection wipes, requiring no mixing or measuring.
- + **Proven efficacy:** effective against Coronavirus and Norovirus in as little as 30 seconds.
- + **Safe to use:** alcohol free, bleach free and PHMB free, causing no skin irritation or sensitisation.
- + **Improved efficiencies:** one product to clean and disinfect, providing you with consistent and reliable results.
- Maximum flexibility: suitable for any healthcare environment available in flow wraps, canisters and buckets.
- + **Optimal protection:** a quaternary ammonium based formulation which can be used on non-invasive medical devices; non-porous hard surfaces.
- + **Cost effective:** using the same solution for cleaning and disinfecting saves you time and money.





## **Product Efficacy**

SANI-CLOTH® AF	TEST METHOD	ORGANISM	CONTACT TIME*	
Bacteria	EN 16615	Pseudomonas aeruginosa	<b>60</b> sec (c)	<b>2</b> min (d)
	EN 16615	Staphylococcus aureus	<b>60</b> sec (c)	<b>2</b> min (d)
	EN 16615	Enterococcus hirae	<b>60</b> sec (c)	<b>2</b> min (d)
	EN 13727	Pseudomonas aeruginosa	<b>30</b> sec (c)	
	EN 13727	Staphylococcus aureus	<b>30</b> sec (c)	
	EN 13727	Enterococcus hirae	<b>30</b> sec (c)	
Yeasts	EN 16615	Candida albicans	<b>60</b> sec (c)	<b>2</b> min (d)
	EN 13624	Candida albicans	<b>30</b> sec (c)	
	EN 13624	Candida auris	<b>30</b> sec (c)	
Enveloped Virus	EN 14476	Vaccinia Virus	<b>30</b> sec (c)	<b>2</b> min (d)
	EN 14476	Corona,HIV, Hep B & C, SARS, Flu	<b>30</b> sec (c)	<b>2</b> min (d)
Non-enveloped Virus	EN 14476	Norovirus	<b>30</b> sec (c)	<b>60</b> sec (d)
ТВ	EN 14348	Mycobacterium terrae	<b>2</b> min (c)	

## Point of Use Accessories



#### Wall bracket for flow wraps Product Code: A003GRN

Made with antimicrobial properties proven to reduce the growth of bacteria by up to 99.99%.



#### **Bracket for canisters** Bracket: A001GRN

Canister brackets affix securely to the wall, holding the canister in place at its location.

**Pole Adapter** Product Code: A002POL

Pole brackets make it possible to affix the canister brackets on to portable equipment such as BP stands.

\*Clean Conditions (c) Dirty Conditions (d)



#### I Am Clean Sticky Notes Product Code: A005SNT

Recyclable sticky notes which can be used to easily and effectively identify equipment or surfaces that have been cleaned.



#### **Educational wall charts** and signage

Product evaluations, audits, educational wall charts & signage, that can be personalised for your specific needs.

## **Range Details**

	REORDER NO	WIPE COUNT	WIPE SIZE	QTY PER CASE	CLASSIFICATION
225ct Bucket	EAF1110	225	270mm x 267mm	1	Medical Device Class IIa
200ct Flow Wrap	EAF1130	200	267mm x 210mm	6	Medical Device Class IIa
100ct Large Canister	EAF1120	100	200mm x 195mm	6	Medical Device Class IIa
40ct Flow Wrap	EAF1140	40	200mm x 200mm	24	Medical Device Class IIa

## Other formulations available:



### Find out more about how our Layered Approach to Infection Prevention can help you.

PDI EMEA, Pywell Road, Willowbrook Est. Industrial Estate, Corby - Northants, NN17 5XJ, UK

🕓 +44 (0) 8081 697 945 🖂 contact@pdi-emea.com 🌐 wearepdi-intl.com in PDI International



PDI\_Intl