

Sani-Cloth® Chlor: Instructions for Use

Two in one sporicidal cleaning & disinfectant wipes

Please follow local healthcare policy for personal protective equipment, hand hygiene & waste disposal guidelines



Preparation



Dispense



- + Dispense the wipe at a 45° angle, pulling away from you.
- + Close the lid when not in use.

Clean



- + If the surface is visibly soiled, clean the surface following an S-shaped motion from top to bottom.
- + Use additional wipes as needed.
- + Use a fresh wipe for each new surface.

Disinfect



- + Using a fresh wipe, disinfect the surface in an S-shaped motion from top to bottom, avoiding cross-contamination.
- + Use a fresh wipe for each new surface.

Contact Time



- + Ensure the surface remains visibly wet for 2 minutes for complete disinfection.
- + Use an additional wipe if necessary.
- + Allow to air dry.

Disposal



- + Dispose of wipe after a single use.
- + Packaging is recyclable



Other information:

- + Store upright in a cool dry place
- + Avoid contact with eyes, skin & clothes
- + Refer to expiry date and POA on packaging



For more information, write to us: contact@pdi-emea.com

+ 44 (0) 8081 697 945

contact@pdi-emea.com

wearepdi-intl.com

 PDI International


BE THE DIFFERENCE®